# VivaStyle®

Instructions for Use for Dentists

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For dental use only.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in USA

Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

Distributed by:

Ivoclar Vivadent Inc., 175 Pineview Drive, Amherst, NY 14228, USA

CE 0123

562011/0605

ivoclar  
vivadent®  
clinical

English

## Instructions for Use for Dentists

### Description

VivaStyle tooth whitening gel is used for the lightening of discoloured teeth. The gel is applied by means of a custom-fitted tray.

### Composition

Carbamide peroxide (10% or 16%), glycerine, buffered polycarboxylic acid, and peppermint oil.

### Indication

- Whitening of discoloured vital teeth
- Whitening of discoloured non-vital teeth

Individual teeth or several teeth in a row can be whitened.

Discolorations can be caused by:

- medications, such as tetracycline, minocycline
- fluorosis
- food-related stains
- age-dependent changes

VivaStyle tooth whitening gel should be applied before the placement of new tooth-coloured restorations, as the tooth whitening gel does not whiten any existing dental restorations. The treatment with VivaStyle should be completed two weeks before the placement of a new restoration. Existing restorations may not match the lighter shade of the teeth after the treatment and may need to be replaced.

### Contraindications

- VivaStyle gel should not be used by:
- pregnant or lactating women
  - children

- patients who are heavy smokers
- patients who consume large quantities of alcoholic beverages
- patients known to be allergic to any of the material's ingredients
- in the presence of teeth which are cervically exposed
- in the presence of untreated primary or secondary caries
- in the presence of crowns and bridges with marginal gaps

### **Side effects**

In individual cases, temporary irritation of the mucous membrane or sensitivity of the teeth may occur. These side effects usually subside within a short time after the treatment is finished or discontinued.

### **Application procedure**

The dental professional examines the patient's medical and dental history, determines the indication, prescribes the length of the treatment and monitors the whitening treatment. The patient applies VivaStyle gel at home, after having been instructed by the dental professional.

### **Step-by-step instructions**

#### **Pretreatment**

Prior to the application of VivaStyle tooth whitening gel, the patient's medical and dental history should be carefully examined and professional tooth cleaning carried out.

#### **Tray fabrication**

A properly fabricated tray will improve the effectiveness of the whitening process. Well defined trays will improve patient comfort and reduce the chance of irritation of the mucous membrane.

1. Take impression, fabricate a stone model and trim it for optimal lingual adaptation.
2. Heat tray material on a vacuum former until it sags 1 to 2 cm (0.4 to 0.8 inch), adapt to model and place under vacuum for 30 seconds.
3. After cooling, remove tray from stone model and remove excess material.
4. Carefully scallop the tray about 0.5 mm – 1 mm short of the gingival margins, using either fine scissors or a blade.
5. Place tray back on the stone model to assess accuracy of fit. Final shaping and defining of periphery may be accomplished with a coarse acrylic polisher.

### **Instructing the patient**

Careful instruction of the patient is fundamental to the safe, effective application of VivaStyle tooth whitening gel at home.

1. Review each individual step of the application procedure in detail with your patient, using the enclosed information on VivaStyle.
2. Check for an accurate fit of the tray.
3. Using a VivaStyle gel syringe, show the patient how to fill the tray correctly. It is important not to overfill the trays. The calibration markings on the syringe should be used as guide when extruding the material.
4. Show the patient how to insert the tray into the mouth. Explain the proper procedure for removal of any excess tooth whitening gel.
5. Determine how long the patient should wear the tray. One hour a day is usually sufficient. Depending on the individual requirements of the patient, a longer wear time may be prescribed.
6. Explain the proper removal and cleaning procedures for the tray. The tray should be stored in the enclosed tray case.

## Notes

- Provide the patient only with as many syringes as are actually needed for the treatment.
- The patients should be reminded of the following points in particular:
  - Avoid the consumption of citrus fruits and fruit juices during the treatment, as they can promote sensitivity.
  - Avoid, or at least try to minimize, the consumption of coffee, tea, red wine and colas and smoking during the treatment to prevent the chance of discoloration occurring in the teeth.
  - Overfilling the tray with tooth whitening gel does not result in faster or better results, and may cause sensitivity.
  - Patients should call their dentist if they have questions about the treatment or if unpleasant sensitivity occurs.

## Warning

VivaStyle gel contains carbamide peroxide and is an irritant. Avoid contact of the gel with the skin, mucosal tissues and eyes. If contact occurs, wash the areas affected with copious amounts of water. In case of contact with the eyes, immediately rinse with copious amounts of water and seek medical advice.

## Storage

- Store material at 5–22 °C (41–72 °F).
- Protect VivaStyle syringes from heat and sunlight.
- Do not freeze VivaStyle syringes.
- See date of expiration for shelf life; observe the information on temperature.
- Do not use after the date of expiration.

**Keep out of the reach of children!  
Only for the described use in the oral cavity!**

**Date information prepared:**  
11/2003

**Ivoclar Vivadent AG**  
FL-9494 Schaan/Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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## Ivoclar Vivadent UK Limited
## Ivoclar Vivadent, Inc.
## Dentigenix Inc.