# VivaStyle® 30%

## Instructions for Use

## Gebrauchsinformation

## Mode d'emploi

## Istruzioni d'uso

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- Tooth whitening gel
- Gel zur Zahnaufhellung
- Gel de blanchiment dentaire
- Gel dentale sbiancante
- Gel para el blanqueamiento dental
- Gel para clareamento dental
- Tandblekningsgel
- Gel til blegning af misfarvede tænder
- Hampaiden valkaisugeeli
- Gel til bleking av tenner
- Gel voor het witter maken van tanden en kiezen
- Ζελέ λεύκανσης δοντιών

For dental use only.

Made in USA

Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

Distributed by:

Ivoclar Vivadent Inc., 175 Pineview Drive, Amherst,  
NY 14228, USA

CE 0123

579743/0907

## Bruksanvisning

## Brugsanvisning

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## Bruksanvisning

## Productinformatie

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## Instructions for Use

### Description

VivaStyle tooth whitening gel is used for the lightening of teeth. The gel is applied by means of a custom-fitted tray. The treatment is carried out by the dental professional in the dental office.

### Composition

Propylene glycol, carbamide peroxide (30%), aqua, carbomer, peppermint oil.

### Indication

- Whitening of discoloured vital teeth
- Whitening of discoloured non-vital teeth

Individual teeth or several teeth in a row can be whitened.

The whitening treatment in the dental office can be combined with a follow-up at-home treatment by the patient. A lower concentration of VivaStyle gel is used for applications at home.

Discoloration can be caused by:

- medications, such as tetracycline and minocycline
- fluorosis
- food-related stains
- age-dependent changes

### Contraindication

VivaStyle gel should not be used by:

- pregnant or lactating women
- children
- patients who are heavy smokers
- patients who consume large quantities of alcoholic beverages
- patients who are known to be allergic to any of the material's ingredients

- patients with teeth which are cervically exposed
- patients who have untreated primary or secondary caries
- patients who have restorations with marginal gaps

### Side effects

Temporary irritation of the mucous membrane or sensitivity of the teeth may occur. These side effects usually subside within a short time after the treatment is finished or discontinued.

### Application procedure

The dental professional examines the patient's medical and dental history, determines the indication, prescribes the length of the treatment, applies the whitening gel and monitors the whitening process.

### Step-by-step instructions

#### Pretreatment

Prior to the application of VivaStyle tooth whitening gel, the patient's medical and dental history should be carefully examined and professional tooth cleaning carried out.

#### Tray fabrication

A properly fabricated tray will improve the effectiveness of the whitening process. Snugly fitting trays will improve patient comfort and reduce the chance of irritating the mucous membrane. It is advisable to use VivaStyle tray material only.

1. Take impression, fabricate a stone model and trim it for optimal lingual adaptation.
2. Heat tray material on a vacuum former until it sags 1 to 2 cm (0.4 to 0.8 inch), adapt to model and place under vacuum for 30 seconds.

3. After cooling, remove tray from stone model and remove excess material.
4. Carefully scallop the tray about 0.5 mm short of the gingival margins, using either fine scissors or a blade.
5. Place tray back on the stone model to assess accuracy of fit. Final shaping and defining of periphery may be accomplished with a coarse acrylic polisher.

### Application

1. Check for an accurate fit of tray.
2. Remove the cap from the resealable application tip of the syringe by twisting it off. Place a small amount of gel toward the facial side of each tooth space in the whitening tray. Use the calibration markings on the syringe as a guide when extruding the material; a syringe usually contains enough gel to fill approx. three upper and lower trays.  
**Note:** Do not apply too much gel into the trays. Overfilling does not result in faster or better results, and may cause sensitivity. Reseal syringes that still contain whitening gel by placing the cap back on the application tip.
3. Place the tray with gel over the patient's teeth.  
**Note:** Do not squeeze the tray when inserting it.
4. Remove excess gel by means of a cotton roll or tissue – prevent the patient from swallowing the material!
5. Determine how long the patient should wear the tray. Twenty to thirty minutes are usually sufficient. Depending on the individual requirements of the patient, the whitening procedure may be repeated.  
**Note:** The patient must not eat, drink or smoke while wearing the tray.
6. Remove the tray after the whitening process has been completed.

7. Remove any remaining gel and rinse thoroughly.
8. Clean the tray with a toothbrush under running water.
9. Repeat step 2 to 8 for further applications.

### Tip

Tooth whitening with VivaStyle 30% can be used as initial whitening treatment. The whitening procedure can be continued by the patient at home using a lower concentration of VivaStyle whitening gel.

### Notes

- The Patients should be reminded of the following points in particular:
  - Avoid, or at least try to minimize, the consumption of coffee, tea, red wine, colas and smoking to prevent the chance of discoloration occurring in the teeth.
  - Avoid the consumption of citrus fruits and fruit juices on the day of the treatment, as they can promote sensitivity.
- Overfilling the tray with tooth whitening gel does not result in faster or better results, and may cause sensitivity.
- VivaStyle tooth whitening gel should be applied before the placement of new tooth-coloured restorations, as the tooth whitening gel does not whiten any existing dental restorations. The treatment with VivaStyle should be completed at least two weeks before the placement of a new restoration. Existing restorations may not match the lighter shade of the teeth after the treatment and may need to be replaced.

### Warning

VivaStyle gel contains carbamide peroxide and is an irritant. Avoid contact of the gel with the skin, mucosal tissues and eyes. If contact with the eyes occurs, wash the areas affected with copious amounts of water and seek medical advice.

### Storage

- Store material at 2–8°C (36–46°F).
- Protect VivaStyle gel from heat and sunlight.
- Do not freeze VivaStyle syringes.
- See date of expiration for shelf life.

**Keep out of the reach of children!  
For dental use only!**

**Date information prepared:**  
09/2007

**Ivoclar Vivadent AG**  
FL-9494 Schaan/Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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## **Ivoclar Vivadent AB**
## **Ivoclar Vivadent UK Limited**
## **Ivoclar Vivadent, Inc.**